

Memo No.2018-037Date:21-Aug-2018Memo To:ClientsRe:Tryptase, Serum

Please note the following test information for Tryptase, Serum, effective September 1, 2018. This change is necessary because the test will be performed at a new test site.

Specimen:	1.0 mL serum – either no additive or SST. Plasma (Heparin or EDTA) also acceptable.			
Timing of Collection:	To assess anaphylaxis, collect specimen between 15-180 minutes after suspected anaphylactic event. To assess systemic mastocytosis or mast cell activation syndrome, collect the specimen at any time.			
Method:	FEIA	TAT:	12 days	
Test Site:	Laboratory Reference Centre, Hamilton			
Specimen Handling:	Separate ASAP. Store and send frozen. Avoid repeated freeze/thaw cycles.			
Stability	Frozen: 30 days or Refrigerated: 7 days			
Reference Range:	Tryptase, serum: 3.8-11.4 ug/L			

For HL7 Interfaced Clients:				
REF CODE: TRYPHAM	RES CODE:	62779	Units: ug/L	

If you have further questions, please contact Client Care at (416) 422-3000 Ext. 300 or info@ICLabs.ca or you may contact me directly.

Laurie Sweeting MPA, BHA, MLT Director, Laboratory Operations (416) 422-3000 Ext. 225 LaurieSweeting@ICLabs.ca

Want to receive updates by e-mail? Please contact Client Care info@ICLabs.ca

ICL is the only Not-For-Profit dedicated broker of lab diagnostic tests in Canada

57 Gervais Drive, North York, ON M3C 1Z2

(416) 422-3000 Ext. 300 info@ICLabs.ca